

# PeriGen, Inc.

---

PeriCALM<sup>®</sup> Tracings,  
PeriAuth ver.02.02.01

## Release Notes

Record ID: CALM 031802 – DMR-2862

Revision: 01.00


Issued: 2017-11



## Copyright notice

Copyright © 1997-2017. PeriGen, Inc. All rights reserved.

## Publication date

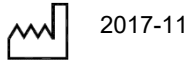

## Product

PeriCALM Tracings, PeriAuth 02.02.01

## Publication number

CALM 031802 – DMR-2862

## Manufactured by



PeriGen Solutions Ltd.  
Sderot Nim 2,  
PO Box 110,  
Rishon LeTzayon, 7510002,  
Israel

## Customer Support

(+1)-866-321-6788, (+1)-888-866-5339, [support@perigen.com](mailto:support@perigen.com)

## Trademarks

PeriCALM Tracings, PeriCALM Curve, PeriCALM Patterns and PeriWatch HUB are trademarks of PeriGen Incorporated. All other product and brand names are trademarks or registered trademarks of their respective companies.

## Intended Use

PeriCALM Tracings is intended for use as a central monitoring system with signal analysis and display functions and remote repeaters in the perinatal clinical environment. It interfaces with standard fetal and maternal monitors and provides fetal surveillance, monitoring of labor progress, charting, reporting and data analysis, display and archiving functions.

CAUTION: Federal law restricts this device to sale by or on the order of a physician.



Please refer to the User Guides prior to the first use.  
Rx only.

Various aspects of the PeriCALM software suite are subject to issued and pending patents in several jurisdictions. Issued patents include:

USA	6,907,284
USA	7,113,819
USA	6,423,016
European Patent	1,505,903
European Patent	1,289,416
Canada	2,311,029
Canada	2,384,516
Canada	2,379,733

## Table of Contents

<b>1</b>	<b>Enhancements .....</b>	<b>4</b>
1.1	PeriAuth database performance improvements .....	4
1.2	PeriAuth configuration diagnostic tools .....	4
<b>2</b>	<b>Fixes.....</b>	<b>4</b>
2.1	PeriAuth made less sensitive to Citrix configuration issues .....	4

## 1 Enhancements

---

### 1.1 PeriAuth database performance improvements

Several optimizations were made to the PeriAuth database in order to improve performance, i.e. shorten query times and reduce CPU.

### 1.2 PeriAuth configuration diagnostic tools

A new database view called PeriAuthSummaryFlat\_V was created in order to provide insight to PeriAuth services' configuration and status.

The view lists the PeriAuth running server, version, psf configuration, configuration date and whether it is on a citrix server.

The data in this view will be populated starting with this release of PeriAuth, so that the view can now be used for diagnosis of PeriAuth configuration issues.

## 2 Fixes

---

### 2.1 PeriAuth made less sensitive to Citrix configuration issues

Behavior of PeriAuth in a Citrix environment was fixed so that now user authentication will be available even if Single Sign On does not work due to Citrix configuration issues. Previously, the red icon would appear and user authentication would be disabled, when using PeriCALM Tracings on a Citrix workstation without having correctly installed PeriAuth on the Citrix server. PeriAuth is now less sensitive to this situation and will support user authentication regardless of configuration errors preventing Single Sign On.